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Herstellerkontakt

Carealytix Digital Health GmbH
Hohendilching 3
83626 Valley
Germany

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Mizu (ENG)

Information about the Product

Mizu is a native companion app designed to assist individuals in documenting and managing their health while living with chronic kidney disease (CKD) across various stages, including early-stage CKD, dialysis, and post-kidney transplant care.

The app enables users to take an active role in managing their health by offering tools for documenting relevant health data. Mizu supports the observation of changes in health parameters over time, allowing users to gain insights into their condition. The documented information can be shared with healthcare providers to support informed discussions and decision-making.

For users at any stage of CKD, including those undergoing dialysis or living with a kidney transplant, Mizu provides features to document treatment-related information, lifestyle factors, and key health metrics. This allows users to observe trends over time, supports adherence to prescribed therapies, and offers a comprehensive overview of their health status.

1. Intended Purpose / Indication

Intended purpose

Mizu helps patients suffering from different stages of chronic kidney disease (CKD), end-stage renal disease, or kidney transplant recipients to calculate their BMI.

The application supports patients in understanding the importance of their current body composition based on their disease stage by:

- Providing a BMI calculator (calculation of BMI based on height and weight input),
- Educating patients on proposed target corridors for their current disease stage,
- Improving self-management by helping patients understand what they can do themselves to optimize their personal BMI based on calculated values.

Mizu is not intended for diagnosis, treatment, or medical decision-making, but serves as a patient-centered support tool for disease management.

Medical Indications

Mizu is designed for adult CKD patients (18+ years old) who want to manage their health parameters and body composition independently.

The application is suitable for patients diagnosed with the following ICD-10 codes:

- **N18.1** - Chronic kidney disease, stage 1
- **N18.2** - Chronic kidney disease, stage 2
- **N18.3** - Chronic kidney disease, stage 3
- **N18.4** - Chronic kidney disease, stage 4
- **N18.5** - Chronic kidney disease, stage 5
- **N18.9** - Chronic kidney disease, unspecified
- **N19** - Kidney failure
- **Z94.0** - Kidney transplant status

The application is suitable for all genders and is intended exclusively for adults aged 18 and older.

Contraindications: None

Warnings: None

Usage Requirements

Mizu can be used as a mobile application on smartphones and tablets with internet access. To ensure proper functionality, the app requires a supported operating system. On Android devices, the app supports version 6 or higher, while on iOS devices, version 14 or higher is required. For the best experience, it is recommended to use the latest available version of the operating system.

The app is designed for independent use by adults aged 18 and older and is suitable for all genders. Users should feel comfortable using mobile applications and have basic experience navigating apps on smartphones or tablets. No prior medical knowledge or experience with health-tracking applications is necessary.

Mizu also offers offline functionality, allowing users to continue documenting their health parameters without requiring a continuous internet connection. All data entered while offline will be automatically synchronized once an internet connection is restored, ensuring seamless and uninterrupted documentation of health information.

2. Description and Functionality of the App

Type and Duration of Use

Mizu is designed as a mobile health application for documenting and observing health-related information in individuals with chronic kidney disease (CKD). The app can be used flexibly on a daily basis, allowing users to document health parameters such as blood pressure, weight, and lab results according to their own schedule. The duration of use is not limited, as Mizu is intended to be a long-term tool for continuous health observation.

Users can personalize their routines by documenting specific health data relevant to their CKD stage. The flexibility of the app allows users to adapt their documentation practices to their lifestyle and treatment needs. The observation of health trends over time offers users a clear understanding of their condition and supports discussions with healthcare providers.

Frequency of Use

Mizu can be used as frequently as needed, depending on the user's personal health management goals. Regular documentation, such as daily logging of weight or weekly observation of blood pressure readings, is recommended to ensure consistent monitoring of health parameters over time.

The BMI calculator can be used whenever there are changes in height or weight. Users undergoing dialysis or post-transplant care are encouraged to document treatment-related data after each session to maintain a complete and up-to-date overview of their health progress.

Features

Mizu provides users with a variety of features designed to support the observation and documentation of their health parameters. These features include:

- A BMI calculator for calculating and observing body mass index based on height and weight inputs,
- Tools for documenting vital signs, lab results, and other CKD-related health parameters,
- A customizable medication reminder system to support adherence to prescribed treatments,
- A comprehensive nutritional information library tailored to CKD dietary guidelines,
- A global directory of dialysis centers to help users find nearby facilities, particularly useful during travel,
- Detailed documentation tools for dialysis treatments, including the recording of values such as fluid removal, blood pressure, and other important health parameters.
- The ability to export documented data in PDF format or share it directly with healthcare providers via interfaces with third-party systems used by the latter to enable professional review.

Clinical Benefit

Mizu offers a convenient way for individuals with chronic kidney disease (CKD) to calculate and observe their Body Mass Index (BMI). By providing users with this functionality, the app helps guide day-to-day decisions related to managing their kidney health.

By regularly observing changes in their BMI, users can gain valuable insights into how lifestyle habits, such as diet and physical activity, may affect their overall health. This information can also support more informed conversations with healthcare providers, helping users make adjustments that align with their personal health goals and treatment plans.

Technical Security

Mizu uses comprehensive measures to protect user data and complies with all applicable data protection laws (see Section 5: Data Privacy). However, using the app in unsecured environments can increase the risk of unauthorized access. This risk is particularly relevant when using public devices, unsecured networks, or connections monitored by third parties, which are beyond the control of the app.

If you encounter technical issues or have difficulty accessing the app, the Carealytix Support Team is available via email at info@mizu.com (response time within 24 business hours on weekdays).

To ensure the secure use of Mizu, please follow these recommendations:

Device Security

To ensure the secure use of Mizu, activate protective features such as a password, PIN, or biometric authentication (e.g., fingerprint or facial recognition). The app supports devices running operating systems that meet the minimum version requirements of the Apple App Store and Google Play Store—specifically, iOS 14 or higher and Android 6 or higher.

Avoid using modified devices such as jailbroken iOS devices or rooted Android systems, as these can compromise the app's security features. To maintain device security, make sure your operating system is up to date and that all applications, including antivirus software, are regularly updated. Enable automatic security updates where possible and only download applications from official sources like the Apple App Store or Google Play Store.

Protection of Sensitive Data

Sensitive information should be protected from unauthorized access. Avoid taking screenshots or screen recordings of health-related content within the app to prevent accidental sharing of personal data. Be cautious when saving or transferring exported health data and ensure any data-sharing is done securely. Sensitive data may also remain visible when switching between applications, so it is recommended to fully close the app after each use to minimize security risks.

Authentication and Security

Mizu uses a secure login system based on an email address and password combination. Mizu does not include an automatic logout feature after periods of inactivity. To ensure the protection of sensitive data, manual logout is recommended after each session, particularly when using shared devices or unsecured networks. Users can update their login credentials and manage security preferences at any time through the app's settings menu.

App Permissions

To enhance functionality, Mizu may request permission to access certain device features, such as sending notifications for medication reminders or accessing the device's location for finding nearby dialysis centers. While granting permissions optimizes the app's performance, Mizu remains functional with limited features if permissions are declined.

Interoperability

Mizu supports interoperability for selected data through the internationally recognized FHIR standard (Fast Healthcare Interoperability Resources). This allows specific health data to be securely shared with healthcare providers and integrated into electronic health record (EHR) systems. Only selected data categories are available for export via FHIR, and users have full control over which information is shared.

3. Developer Qualifications

Mizu is developed by a multidisciplinary team of experts in nephrology, digital health, software engineering, and regulatory compliance. The app is created in collaboration with healthcare professionals experienced in managing chronic kidney disease (CKD), end-stage renal disease, and post-transplant care.

Medical professionals contributed to defining the app's clinical relevance, while software engineers and UX designers ensured that the app is user-friendly and intuitive. Data protection specialists ensure compliance with applicable data security and privacy regulations.

User feedback plays a key role in refining Mizu's features. Usability studies involving individuals with CKD and ongoing input from healthcare professionals help ensure continuous improvement and quality assurance.

4. Patient Safety

Contraindications

There are no known medical contraindications for using Mizu. The app is designed exclusively for adults aged 18 years and older who have been diagnosed with chronic kidney disease (CKD), end-stage renal disease, or have undergone a kidney transplant.

Warnings

There are no known warnings associated with the use of Mizu. The app is intended solely for documenting and observing health data and should not be used as a substitute for professional medical advice, diagnosis, or treatment.

Side Effects

While using the app to document changes in health status, some users may experience emotional discomfort due to reflecting on their condition. If this occurs, seeking support from a healthcare professional is recommended.

There are no known further side effects associated with using Mizu. However, incorrect or inconsistent documentation of health data could result in misinterpretation of observed health trends. Users should ensure that the data entered into the app is accurate and regularly updated.

Interaction Risks

There are no known interactions between the use of Mizu and other medical devices, treatments, or medications. However, users should inform their healthcare providers about their use of the app to ensure it complements their existing treatment plans.

5. Data Privacy

Mizu collects only the data that is strictly necessary for the proper functioning of the app and for supporting users in documenting and observing their health parameters. All legal data protection requirements are fully met, and the app complies with the provisions of the General Data Protection Regulation (GDPR) and applicable national data protection laws.

All health data documented within the app is encrypted and stored securely to prevent unauthorized access. For detailed information on how personal data is processed and protected, users can refer to the full privacy policy available at <https://www.mizu-app.com/en/data-privacy>.

Contact for Data Protection Inquiries

If you have any questions regarding data privacy or the processing of personal data, you can contact our Data Protection Officer:

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Email: info@qur.digital

6. Costs

Mizu is available free of charge for all users. There are no additional costs associated with downloading or using the app.

7. Integration into Healthcare Services

Mizu is designed to complement existing healthcare services by allowing users to document and observe their health data independently. The app provides features that help users stay informed about their health status and share relevant information with their healthcare providers.

Data Export and Sharing

Mizu allows users to export specific health data in formats that are compatible with electronic health record (EHR) systems. This enables seamless sharing of documented information with healthcare providers for more informed treatment discussions and improved care coordination.

FHIR Compatibility

For selected data categories, Mizu supports interoperability through the internationally recognized **FHIR** (Fast Healthcare Interoperability Resources) standard. This allows users to securely share specific health data with healthcare providers, ensuring compatibility with most modern EHR systems. Users maintain full control over which data is shared and when.

Supporting Patient-Provider Communication

By providing users with an organized overview of their health parameters, Mizu facilitates clearer communication between patients and healthcare professionals. This supports personalized treatment strategies and ensures that healthcare providers have access to up-to-date information when making decisions about care.

8. Important User Information

Reporting Serious Incidents

If you experience a serious incident related to the use of Mizu, please report it promptly. Incidents should be reported to both the manufacturer and the relevant authority in your country.

To report an incident, please contact:

Manufacturer Contact

Carelytix Digital Health GmbH
Hohendilching 3
83626 Valley

Germany

Email: info@mizu.com

Requesting a Paper Version of the Instruction for Use (IFU)

You can request a printed version of the Instructions for Use at no additional cost. Please send an email to info@mizu.com to request a paper copy. The printed version will be provided within 7 calendar days of receiving your request.

The printed version of the IFU will be made available free of charge and delivered promptly, ensuring all necessary information is accessible.

9. Manufacturer and Product Information

Manufacturer

Carealytix Digital Health GmbH
Hohendilching 3
83626 Valley

Information Status

2025-03-03

10. Symbol Legend

Symbol	Definition
	Medical Device (ISO/DIS 15223-1:2020, Reference no. 5.7.7) This is a medical device.
	CE Marking (Communauté Européenne) Product traded in the extended internal market of the European Economic Area (EEA), meeting high standards of safety, health, and environmental protection.

	Consult Instructions for Use (ISO 7000-1641) The user must also consult the instructions for use.
	Manufacturer (ISO 7000-3082) Name and address of the medical device manufacturer.
	Date of Publication (ISO 7000-2497) Date on which the medical device was officially released.
	Warning (ISO 7000-0434A) Warnings and necessary precautions.

Version: 1.0.0